

AXKID ONE 2 REHA

Functions (selection)



Quick & safe installation in 30 seconds – Simply click the seat into the vehicle's ISOFIX-Connectors, unfold the stand base and fix the automatic rebound bar. Done!




Lateral trunk pads can be velcroed precisely to the large velour upholstery for better upper body support



Directed backwards as a safe reboarder – the child's head is optimally protected in the event of an impact



Central wheel for adjusting the seat angle by up to 42 degrees, ideal for various activities such as playing, resting or sleeping

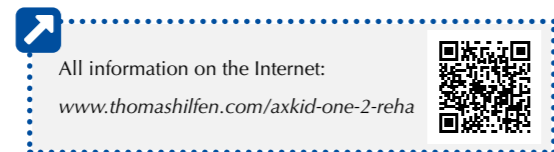

Features



Removable abduction block for a good sitting position, prevents the legs from rolling over



Side impact protection (ASIP) absorbs energy in the event of side collisions. Easily mounted on right or left side.



Headrest that grows with the child, height-adjustable in 12 positions at the back



The sliding seat technology and the Sleep Well System make it easy to adjust the seat to the child's needs

Accessories



ADDITIONAL HEAD PADDING provides better support for the child's head thanks to the additional padding insert




ANTI-ESCAPE-VEST prevents the child from squirming out of the belt



CAR SEAT PROTECTION MAT with back protects the seat cushion and offers three practical compartments at the front for toys or similar

AXKID ONE 2 REHA

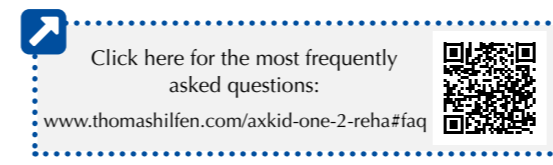

The only Reha ISOFIX child seat that allows directed backwards travel up to a height of 125 cm

Dimensions & weights	Axkid ONE 2 Reha
Seat depth (to crotch strap)	16 cm / 6,3"
Seat depth (to front edge)	28 cm / 11,0"
Seat width	21 - 29 cm / 8,3 - 11,4 "
Backrest height (seat inside)	52 - 63 cm / 20,5 - 24,8"
Backrest width (seat)	30 cm / 11,8"
Min. back height	29 - 42 cm / 11,4 - 16,5"
Max. back height	50 - 63 cm / 19,7 - 24,8"
Head width	16 cm / 6,3"
Shoulder height (with belt guide)	26 - 37 cm / 10,2 - 14,6"
Shoulder width	32 cm / 12,6"
Back angle (max.)	90 - 145°
Seat tilt (basic model)	45° / 65°
Height of trunk pads (upper edge)	28 - 34 cm / 11,0 - 13,4"
Distance trunk pads (upper edge)	18 cm / 7,1"
Abduction block (W x H x D)	7 x 10 x 10 cm / 2,8 x 3,9 x 3,9"
Max. overall dimensions (W x H x D)	44 x 76 x 54 cm / 17,3 x 29,9 x 21,3"
Weight of seat (basic model)	9,9 kg / 21,8 lbs
Body size user	61 - 125 cm / 24,0 - 49,2"
Max. user weight	23 kg / 50,7 lbs

HMV-Nr. 26.11.06.0900



Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstraße 1 · D - 27432 Bremervörde
Phone +49 (0)4761 8860 · Fax: +49 (0)4761 886-19 · info@thomashilfen.de · www.thomashilfen.com



max. 23 kg / 50.7 lbs
61 - 125 cm / 24 - 49.2"

Axkid ONE 2 Reha

THE REAR-FACING CAR SEAT THROUGH CHILDHOOD.

Special car seats for children with special needs.



Status: 2024-03

THOMASHILFEN Axxid ONE 2 Reha

"Axxid ONE 2 Reha is the ultimate ISOFIX child seat that allows children with disabilities **to sit backwards up to a size of 125 cm to reverse.**"

Thanks to its innovative technology, it particularly protects the child's sensitive head and neck while offering **quick and safe installation in just 30 seconds** - without the need for elaborate belts, which avoids dangerous application errors such as incorrect belt routing, twisted belts or belts that do not fit properly, especially when changing vehicles.

It sets a new standard for rehab-child seats and brings safety and flexibility to the everyday life of families."



Pioneer

REHAB CAR SEATS

since
1988

Thomashilfen has the largest range of rehab car seats in the world and offers the right support for all ages and disabilities.



Thomashilfen has been supplying special children with rehab car seats for over 35 years. Today, we also use our expertise to develop our own rehab car seats. They incorporate all our many years of experience in meeting the needs of children and parents.

Gunnar Thomas
Gunnar Thomas, Owner

UNIQUE FEATURES OF Axxid ONE 2 Reha

- Sliding seat technology that gives the best legroom on the market
- Long range safety program – only need for one rear-facing car seat through childhood
- 30 seconds installation guarantee
- Advanced sleep well system with a stepless recline function

- +** REHAB-EQUIPMENT inclusive:
- Abduction block
 - Hip guide pads
 - Head cushion
 - Trunk pads



ASIP side impact protection
➤ Extra stable energy absorbers on the sides significantly reduce the forces of a side impact. Mounted on right or left side.

5-POINT-BELT
Integrated 5-point-belt with central adjuster for quick adjustment of the belt length, practical belt connector prevents the belts from twisting.

Quick fastening without vehicle safety belt.

AT A GLANCE
Features an Easy Installation System with 6 self-explanatory installation steps. The reboarder also has an automatic rebound bar with built-in sound that lets you know when the seat is in the optimum position.

Display of the base:
➔ green: set correctly
➔ red: set incorrectly



DYNAMIC LEGROOM SYSTEM
With its unique leg room adjust can fit from ultra compact mode, requiring just 73 cm of space anywhere up to a maximum of 88 cm with maximum leg room. Noting most children will never require that amount of leg room so the seat will rarely require that amount of room. It is the phenomenal feature of the seat that it has absolute flexibility to fit all family set ups and vehicles regardless of cabin size at the touch of a button.



FORCE ABSORBING TECHNOLOGY
The geometry of the headrest as well as the choice of material is specially designed to absorb the crash forces before reaching the child.

Headrest is adjustable and can be fitted into 12 positions, allowing full ability to meet the height and comfort needs.

PRECISE ADJUSTMENT
To support an improved upper body posture, the lateral trunk pads (1) and hip guide pads (2) can be adjusted to fit your child.



AUTOMATIC REBOUND BAR
Eases the installation and ensures the car seat is firmly installed. It ensures the car seat is secure in the event of any type of car crash, including roll-over accidents. Apart from the safety aspect, the automatic rebound bar provides additional legroom and enables an easy installation under 30 seconds.

SLEEP WELL SYSTEM
The infinitely variable seat tilt of up to 42 degrees can be quickly adjusted to a resting or sleeping position.



Maximum safety – Extra long reversing
The rearward-facing position helps to reduce the strain on the spine when the head is thrown forward during a head-on collision. The force-absorbing technology in the headrest protects the child's sensitive head and neck in the event of an impact.

When the child is sitting in the reboarder, the risk of serious injury is reduced by 92 percent.



The only ISOFIX rehab child seat on the market with i-Size approval up to 125 cm/ 23 kg.

Quick & secure 30 seconds installation.

YouTube

Watch the video:
How to install the
Axxid ONE 2 Reha in the car

