# RECARO

## Sport HERO Reha

DE

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**Safety note:**

Only use the belt extension for children in ECE group II + III (18-36 kg). The extended 5-point positioning belt is NOT a safety-relevant item. In this case the child is restrained by the vehicle's 3-point safety belt. The 5-point belt is only intended for use as a positioning belt to provide additional support for children with a body weight of between 18 and 36 kg (39.7 and 79.4 lb).

First please dismantle the attached 5-point belt. Follow the instructions in your RECARO Sport HERO Reha manual which is stored in your car seat.

Install the extended positioning-belt in the reverse order of removal. Please guide the belt below the bar which is marked in the picture (fig. 1).

To finish the installation please clip the base plate back into place.

After the installation and before using the seat please ensure that:

all removed parts are reinstalled and only the extracted 5-point belt remains.

- the belts are not twisted.
- the belt must fit snug, straight and flat across the child but not be overly tight.
- the shoulder belt energy absorber is reinstalled properly.
- the belt buckle is positioned at the right height and is equipped with the belt buckle energy absorber (accessory).

## **Using the extended 5-point positioning belt for large children (up to 36 kg / 79.4 lb)**

The RECARO Sport HERO Reha child car seat has successfully passed a crash test at Dekra (in Klettwitz) on the basis of the requirements of ECE R 44/04. The RECARO Sport HERO Reha is therefore approved for use as a child restraint system for special needs "S".

To enable your child to use the RECARO Sport HERO Reha for the longest possible time, the standard 5-point safety belt system, including the guide rail, can be removed. This allows the headrest to be adjusted to a higher position. When making this adjustment please observe the instructions contained in the original RECARO Instructions for Installation and Use.

If it is intended to use a 5-point positioning belt for larger children, please observe the following installation instructions for the extended 5-point positioning belt.

First remove the existing 5-point safety belt together with the guide rail (see the notes in the original RECARO Instructions for Installation and Use). However, do not completely remove the shoulder pad, but simply pull the guide rail out from the shoulder pad's fastening straps.

Installation of the 5-point positioning belt is easier when the buckle tongues are latched into the belt buckle. Thread the lap belts downwards from the top, through the outer slots in the seat base, and then feed them through the inner slots back to the top. Now insert the locking plates into the loops (fig. 2).

Next, feed the crotch strap through the slot in the seat base and secure the loop on the end using the associated mounting plate. Now clip the base plate back into position.

Thread the shoulder straps through the slots in the cushion and the seat shell (fig. 3). The shoulder straps are now routed over the seat's body shell without using the guide rail. Engage the shoulder straps into the T-section of the adjuster strap (fig. 4). Finally hook-and-loop-fasten the shoulder pads around the belts. You can push the straps for fastening the shoulder pads to the guide rail into the shoulder pad.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany