ThevoTherapy

Armlehnen

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Armrest

Assembly instructions – Accessory

Accoudoir

Notice de montage – Accessoires

Reposabrazos

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Before mounting the armrests, the lateral plates need to be dismantled. Take off the upholstery, remove the two screws on the inside and remove the plate (fig. 1). Now place the armrests at the outside of the flat plates and mount them with the fillister head screws coming from the inside (fig. 2). Finally reattach the lateral plates and the upholstery.

CAUTION: Make sure that the receivers are tightly fixed.

Height adjustment

The height is adjustable in a 5 cm / 2" range. Size 1 is adjustable from 12 to 15 cm / 4.7 to 5.9" and size 2 from 15 to 20 cm / 5.9 to 7.9". Loosen the bottom knurled screw. Pull the armrest slightly outwards so that the locking pins no longer grip into the holes of the receiver track. Adjust the desired height and plug the receiver track onto the locking pins again. Retighten the knurled screw. (fig. 3)

Angle adjustment

Loosen the upper knurled screw at the grid joint, adjust the desired angle and retighten the screw (fig. 4).

Lateral adjustment of the armrest upholstery

Loosen both screws underneath the upholstery, adjust the desired position and retighten the screws. (fig. 5)

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