Simplex

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Simplex - absenkbare Badeliege

User Manual

Simplex - Bath Chair Lift

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Simplex

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1. General instructions for use

Dear User,

Before you use the bath chair lift, please read these operating instructions carefully. They contain important information that will allow you to handle the chair lift easily and properly.

The chair lift supports handicapped children when bathing and provides safe support at all times.

2. Safety instructions

- Make sure immediately after you unpack the Simplex Bath Chair Lift that it is fit for use.
- Before you use the chair lift, check the connecting cord to lower the chair lift for damage.
- The child should always be secured by the double shoulder harness to prevent the child from falling out, slipping or climbing out of the chair unintentionally.
- Only use the bath chair lift in fixed bath tubs.
- Never leave the child unsupervised in the Simplex bath chair lift.
- Do not exceed the maximum load-bearing weight of 25 kg / 55.1 lb.

3. Installing and removing the bath chair lift

To install the Simplex bath chair lift, simply place it on the sides of the bathtub with both hands.

Important:

When placing the chair lift on the bath, make sure the lifting arm is secured.

To remove the chair lift, first secure the lifting arm. Then remove the chair lift from the sides of the bathtub with both hands.

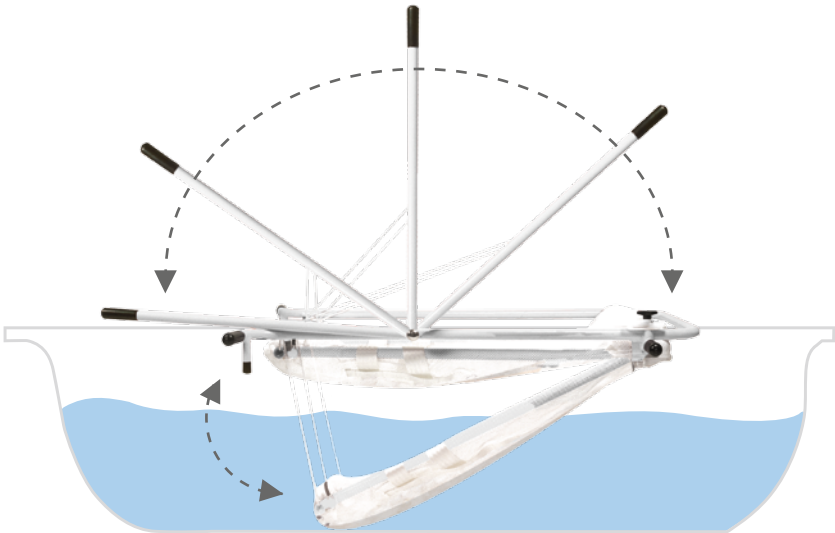
4. Lowering

Lower the Simplex chair lift using the lifting lever. Handling the lever is very simple. Release the lifting arm from the catch as shown in the illustration. Then swing the lifting arm over to lower the chair. Swing the lifting arm back to raise the chair.

ATTENTION: Regularly check the tightness of the knurled screw locking. When the lifting arm is locked, the white safety plastic disk must always be positioned horizontally to the lifting arm tube.

Swing the lifting arm over to lower the Simplex chair lift safely:

5. Cleaning and care instructions

Simplex is made of high quality steel tubing coated with a hardwearing powder enamel finish. The seat, backrest and belt are made of nylon. The powder-coated steel tubing and nylon can be cleaned with commercial soaps (neutral soap). The chair lift can also be disinfected using commercial domestic disinfectants.

6. Technical specifications

Seat width	32 cm / 12.6"
Overall dimensions (l x w x h)	125 x 74 x 15 cm / 49.2 x 29.1 x 5.9"
Bath width	54 - 64 cm / 21.3 - 25.8"
Max lowering height	35 cm / 13.7"
Weight	8 kg / 17.6 lb

7. Maximum load-bearing weight

The max load-bearing weight is 25 kg / 55.1 lb.

8. Warranty conditions

Thomashilfen grants a warranty of 6 months. The warranty covers all claims for workmanship and material defects which impair the product function. Exclusions include any damage caused by improper handling (e.g. overload) and natural wear and tear.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG