EASyS

(DE)

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(GB)

Headrest, flexible

Assembly instructions – Accessory

(FR)

Appuie-tête, flexible

Notice de montage – Accessoires

(ES)

Reposacabezas, flexible

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Headrest flexible, lateral

Item code: 6627

Remove cushion completely before drilling to avoid any damage.

Place the drilling template (page 9) on the rear of the seat unit centrally below the upper edge of the back plate (1).

CAUTION: The upper tube first has to be pushed up. You can do this with a push-button on the right side (when viewed from behind) after the locking screws have been removed from the tube on the rear.

Then drill the four holes in the middle (5 mm diameter).

Now push the two lower screws through the holes, the raised tube on the back is then pushed between the upper and lower pair of screws again. The metal sleeves and mounting support are slid onto the screws from the outside and the screws tightened with the nuts.

CAUTION: The thread for the knurled screw on the mounting bracket must be on the right-hand side (2).

Now place the mounting bracket on the metal plate (3).

Ten pull the protective bag over the bracket, making sure that the top and bottom holes in the bag are directly above or below the opening for the bracket. Push the mounting device with headrest trough the opening and fasten with the knurled screw.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany