trio

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Dear trio-User!

With the trio walker you made a good choice. It will help you to cope with your daily life or during your walking therapy. The walker is designed for the use indoors.

Operating the trio walker is simple and not complicated at all. Width and height can be adjusted without needing any tools. Please read this user manual in order to ensure that the handling of the trio walker is safe, practical and comfortable for you.

If you still have any questions or problems, please call your local dealer or contact us directly.

We want to take this opportunity to draw your attention to our website www.thomashilfen.com, where you can find the latest information and important product safety information, as well as information about any product recalls.

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Table of Contents	Page
Safety checks and maintenance interval	11
General notes regarding safety	11
Purpose and areas of application.....	11
Application risks and contraindications	11
Symbols and labels	12
Specifications and versions	12
Scope of delivery	12
Basic equipment of the walker	13
Setup and assembly	13
Possible settings	13
Product / lot number	14
Cleaning, disinfection and care	14
Warranty terms / CE label	14
Maintenance schedule – annual inspection	15

Safety checks and maintenance intervals

A visual inspection of all the operating controls and fastening screws should be carried out each month by a qualified specialist.

Please observe the following notes regarding safety

- Before using the product for the first time, read the user manual carefully; if you have poor eyesight, have someone read it to you. If the user manual is lost, you can download it at any time in the download area of our website www.thomashilfen.com. The stored PDF document can also be displayed at a larger size.
- The use of the walker by people with limited mobility must always be supervised by a carer to prevent falls!
- Before using the walker all screw connections, especially those for the height adjustment, have to be checked and, if necessary, retightened on a regular basis!
- Always secure the trio walker against rolling when getting in or off! You can lock it by pushing the button on the handbrake. To unlock again, slightly pull the brake arm of the hand brake.
- **CAUTION:** The trio walker is no raising-aid und must not be used as such!
- Please be very careful when passing small obstacles, e.g. doorsteps. There is always a risk for the walker to tip over! Never pass those obstacles reverse.
- Make sure to rest on the entire forearm or to sufficiently support yourself on the armpit support (accessory).
- Always hold on to the front handles.
- Use the walker on even ground only.
- The trio walker should only be used indoors.
- **CAUTION:** Please be careful when adjusting the armrest and armpit support. There is a danger of trapping your fingers.
- **CAUTION:** The armrests can get very hot when exposed to direct sunlight for a longer time. There is a risk of injury in case of direct skin contact. Only use the trio walker when it has cooled sufficiently.
- Please do not overload your trio walker and observe the maximum load capacity (see the specifications).

Purpose and areas of application

Walkers support people with walking disabilities by reducing the strain of the lower extremities. In order to use the walker the user needs to be able to apply their own force and to have a basic ability to stand. The walker provides for a relieving and safe walking training for diseases or injuries which implicate disorders in the walking apparatus and / or coordination and balance problems.

Application risks and contraindications

When used appropriately, risks for the application can be excluded. Contraindications are not known.

Symbols and labels

Sticker	Meaning
	<p>Specification label / loading capacity stickers</p> <p>The specification label is attached to the frame.</p> <p>The specification label includes the name of the manufacturer, the type designation, the maximum loading capacity and the CE label.</p>
	<p>Bar code label</p> <p>The bar code label is attached under the armrests.</p> <p>The bar code label includes the item number, designation, Lot number and production date of the product.</p>

Specifications and versions

	trio Standard	trio A	trio B	trio BA
Width adjustment (armrest)	–	–	400 - 530 mm / 15.8 - 20.9"	400 - 530 mm 15.8 - 20.9"
Height (armrest)	900 - 1300 mm / 35.4 - 51.2"	900 - 1300 mm / 35.4 - 51.2"	900 - 1300 mm / 35.4 - 51.2"	900 - 1300 mm / 35.4 - 51.2"
Overall dimensions (L x B)	900 x 710 mm / 35.4 x 28"	900 x 710 mm / 35.4 x 28"	900 x 710 - 770 mm / 35.4 x 28 - 30.3"	900 x 710 - 770 mm / 35.4 x 28 - 30.3"
max. turning circle	1200 mm / 47.2"	1200 mm / 47.2"	1200 mm / 47.2"	1200 mm / 47.2"
Wheel size	200 x 50 mm / 7.9 x 2"	200 x 50 mm / 7.9 x 2"	200 x 50 mm / 7.9 x 2"	200 x 50 mm / 7.9 x 2"
Weight	21 kg / 46.3 lb	24,4 kg / 53.8 lb	21 kg / 46.3 lb	24,4 kg / 53.8 lb
max. user weight	150 kg / 330.7 lb	150 kg / 330.7 lb	150 kg / 330.7 lb	150 kg / 330.7 lb

The trio walker is available in the following versions:

Artikel-Nr.	Designation	Equipment
2050	trio walker Standard	Walker with armrests
2055	trio walker A	Walker with armrests and armpit supports
2045	trio walker B	Walker with armrests and width adjustment
2040	trio walker BA	Walker with armrests, width adjustment and armpit supports

Scope of delivery

The trio walker is delivered pre-assembled. For transportation the handles are turned over on the underside of the walker's top part. The trio walker is supplied with a user manual. The trio Walkers A and BA are supplied with armpit supports.

Basic equipment

Height adjustable chassis, armrests, two adjustable handles, two lockable brakes, swivel front wheels, Flexel-rubber tires

Setup and assembly

The settings / adjustments must always be made by trained / instructed professionals (such as medical product consultants of specialist medical dealers). No tools are required when using the trio walker for the first time.

For transportation the handles are turned over on the underside of the walker's top part. In order to position them correctly, loosen the screw and turn the handles until they face upwards (fig. 1). If possible, maintain the 90° angle between handle and armrest (both ways horizontally and vertically). Make sure the grid matches up.

Mounting the armpit support (accessory)

Remove the plastic plugs from the frame of the armrests. Now slide the metal adapter through the provided holes as far as it will go (fig. 6) and lock the adapter with the screws supplied. Push the left / right tube of the armpit support through the adapter with the pad facing upwards (fig. 7).

Possible settings

Height adjustment of the armrest

The height of the armrests can be adjusted by loosening the two clamp screws on either side (fig. 2). The height is set correctly when the arms of the user are resting in a 90° angle on the arm supports.

Caution: After changing the position of the armrest do not forget to tighten the clamp screws! To avoid damage, do not tighten the screws with more than 10 Nm! Also pay attention to the maximum possible extension position (marked on the tubes)!

Adjusting the brakes

The brake action can be fine adjusted by tightening the brake cable on the hand brake. By turning the cable tightener to the right, the brake action is reduced (fig. 3).

In addition, the brake can be adjusted at the end of the brake cable (fig. 4) by turning the cable tightener.

Caution: Always make sure that the wheel can only just swivel when the brake is released. Too much space between wheel and brake reduces the brake action!

Width adjustment

If you possess a model with width adjustment, you can change the position of the armrests. Loosen the screws underneath the armrests (fig. 5). Adjust the desired position and retighten the screws.

Correct height adjustment of the armpit supports (accessory)

After having adjusted the armrests correctly, the armpit supports have to be adjusted. The padded top of the armpit support has to be high enough to provide good support for the user of the walker. In order to do this, the user has to stand up straight and relaxed. Unlock the clamping lever (fig. 8) and adjust the height until the supports are located in the armpits.

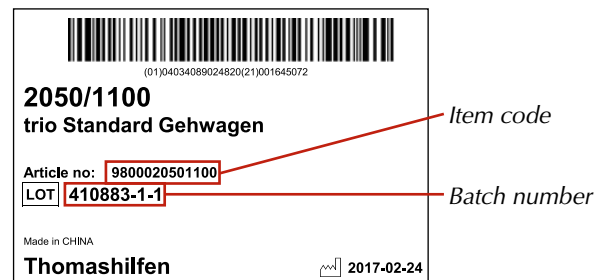
TIP: The required basic height is reached by adjusting the height of the armrest. The fine adjustment is then be made by adjusting the height of the armpit support.

CAUTION: After setting the height, tighten back up the clamping lever! Use at most 10 Nm (hand-tight) for tightening, otherwise damaging could happen! Pay attention to the maximum possible extension position (marked on the tubes)!

Product / Lot number

To obtain the correct spare parts or for any technical callbacks that may be required, we always need you to provide us with the lot number and the precise item number.

The lot number and the item number of the trio walker are located on the bar code label that is attached under the armrest.

Cleaning, disinfection and care

The chassis is made of high-quality steel tubes. These are coated with durable powder stove enameling that can be cleaned with commercial (neutral) soaps. Please do not use any abrasive, aggressive, chloric, or acidic cleaning agents for the trio walker, but detergents suitable for plastic and lacquer.

Disinfection

Clean armrests, handles and armpit supports (accessory) before disinfection. Wipe all parts of the product with a cloth that is damp with disinfectant. Only water-based disinfectants should be used. When disinfecting, the manufacturer's application instructions must be heeded.

Warranty terms / CE label

Thomashilfen offers you a guarantee of 24 months on manufacture and material of the walker. Not included are damages caused by inappropriate use or normal wear and tear. Even after the discontinuation of this model Thomashilfen will stock the corresponding spare parts. This ensures the replacement of parts, e.g. for the reuse of the product, for a long time.

The trio walker meets the requirements of standard DIN EN ISO 11199-3:2005 and is provided with a CE label.

We hope you enjoy your trio walker!

Maintenance schedule – annual inspection / reutilization

Preparation

A user manual must accompany the product. If none is present, please order one from the manufacturer. Familiarize yourself with the functions of the product. If you are not familiar with the product, read the instructions carefully before the inspection. Clean the product before the inspection. If necessary, heed the care instructions and product-specific inspection instructions in the user manual.

Item	Area <input checked="" type="checkbox"/> = Inspection performed	(I = Inspection / R = Reutilization)							
		Function		Damage		Warping		Replace	
1	Wheels	I		I		I		R	
2	Tubes / chassis	I/R		I/R		I/R			
3	Brakes	I/R		I/R		I/R			
4	Armrests	I/R		I/R		I/R			
5	Handles	I/R		I/R		I/R			
	Options								
6	Width adjustment	I/R		I/R		I/R			
7	Armpit supports	I/R		I/R		I/R			
Maintenance was carried out						on:			

Service and repairs to the trio walker may be carried out only by qualified personnel. The user of this rehab aid must ensure that the required inspections / maintenance be carried out regularly and on schedule. If damage is evident, the user must actively inform the qualified personnel and have the damage repaired immediately. Only original spare parts may be used for repairs (the list of replacement / spare parts can be found in the download area on our website www.thomashilfen.com).

Removal / attachment of spare parts must always be performed by qualified personnel only! If it is necessary to return the product to your dealer, please make sure that you pack the walker so that it can be transported without damage.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG